Angelcare®

AC201
AC201-R

Movement & Sound Monitor

Movement & Sound Monitor (Rechargeable)

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TABLE OF CONTENTS

5 Easy Steps to Operate Your Angelcare® Monitor	3
Warnings	4
Instructions	4
Testing Your Angelcare® Monitor	4
Operating tips	4
Troubleshooting	5
Warranty	6
Distributors	6

Figure 1A


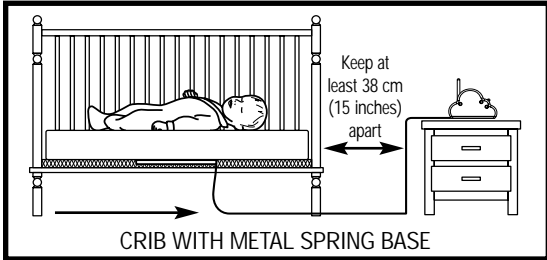
Figure 1B

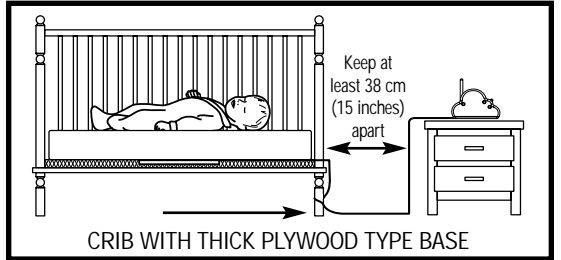SECURE SENSOR PAD CORD TO CRIB LEG USING A TIE WRAP. RUN SENSOR PAD CORD ON FLOOR TO ENSURE CORD IS OUT OF BABY'S REACH.

Figure 2

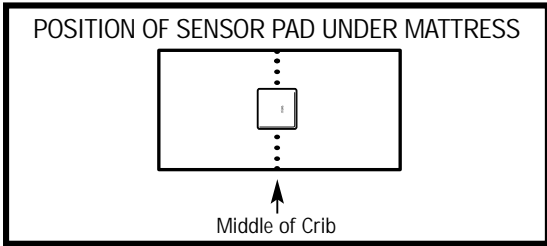
Figure 3

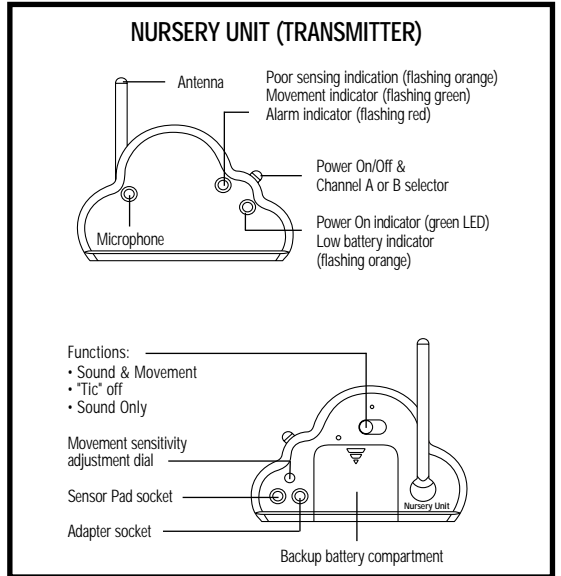
Figure 4

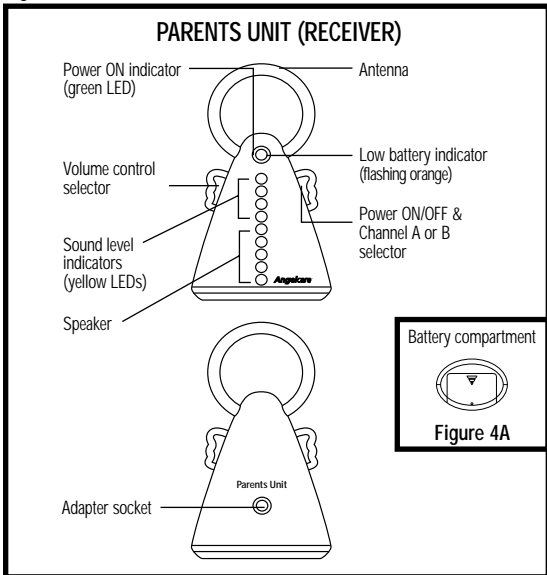
Figure 5


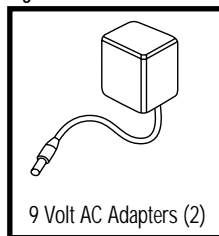
Figure 6

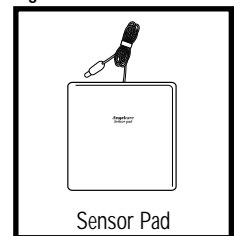5 EASY STEPS TO OPERATE YOUR ANGELCARE® MONITOR

STEP 1. SENSOR PAD

Place Sensor Pad **under** crib mattress, directly under where baby sleeps (Figures 1 and 2). Printed side of Sensor Pad **MUST** face up. **DO NOT** place any bedding or linen between crib mattress and Sensor Pad.

Sensor pad must rest on a **completely flat and rigid surface**.

If your crib has a flexible or uneven base (metal spring base, etc.), place a piece of plywood measuring at least 400 mm X 600 mm X 6 mm (16" X 24" X 1/4") under the sensor pad to create a firm surface.

WARNING: Possible entanglement or strangulation injury. Keep all cords out of baby's reach.

Cribs with metal spring base: Route sensor pad cord through metal spring base in the center of crib (figure 1A).

Cribs with plywood type base: If your plywood base has air holes, route sensor pad cord through air hole. Otherwise, if your plywood base does not have air holes, bring sensor pad cord to end of the crib (figure 1B).

SECURE SENSOR PAD CORD TO CRIB LEG USING A TIE WRAP. RUN SENSOR PAD CORD ON FLOOR TO ENSURE CORD IS OUT OF BABY'S REACH. KEEP NURSERY UNIT AT LEAST 380 MM (15") AWAY FROM CRIB.

STEP 2. NURSERY UNIT (TRANSMITTER)

- Insert 4 AAA batteries into Nursery Unit (for battery backup in case of power failure).

- Connect Sensor Pad to Nursery Unit (Figures 3 and 6).

- On the back of the Nursery Unit, select 1 of the 3 functions of how you want to monitor your baby from the Parents Unit:

Sound & Movement: To hear sounds, see sound lights and hear a "Tic" sound with every movement baby makes.

Tic off: Same as Sound & Movement function **BUT** without the "Tic" sound.

Sound Only: To hear sounds and see sound lights on Parents Unit.

- Connect AC adapter to Nursery Unit and plug adapter into an electrical outlet.

NOTE: The sensitivity adjustment has been preset. **DO NOT ADJUST UNLESS NECESSARY.** If poor capture of movement (orange flashing light) or false alarm occurs, consult the Movement Sensitivity Adjustment section.

STEP 3. PARENTS UNIT (RECEIVER)

Model AC201: Insert 4 AAA batteries into Parents Unit. Or, connect AC adapter to Parents Unit and plug adapter to an electrical outlet.

Model AC201-R: The pre-installed NiMh rechargeable batteries need to be charged for **15 hours continuously before first-time use**. To charge, connect the AC adapter to the Parents Unit. Plug AC adapter to an electrical outlet (Figure 4). The batteries will charge as long as it is connected to a powered AC adapter. The position of the switches will not effect charging.

For longest battery life, recharge the Parents Unit when the orange Low Battery light flashes. Batteries need to be charged for 12 hours continuously.

NOTE: Use only rechargeable batteries in Parents Unit. Do not use Alkaline batteries in Parents Unit.

- Adjust the volume control to the desired level.

STEP 4. TO USE MONITOR

Place baby in crib and switch both units ON by selecting either channel A or B. If reception is not clear, switch to other channel. **Make sure both units are on the same channel.** When removing baby from crib, switch Nursery Unit to the OFF position. If you forget to switch OFF, a "pre-alarm Tic" will sound after 15 seconds as a reminder.

To avoid feedback (whistling sound), keep both units at least 3 meters apart (10 feet).

STEP 5. BATTERY MODE

- When unplugging AC adapter from each unit, **make sure** unit is switched OFF. Otherwise, unit will revert to battery mode.

- The monitor will revert to battery mode during a power failure only if batteries are inserted in both units.

Note:

- Test your Angelcare® Monitor **BEFORE** first-time use and then on a periodic basis.
- When monitor is in use, **DO NOT** use crib mobile or other crib accessories that produce vibrations. Alarm will not sound as monitor is still detecting movement.
- The Angelcare® Monitor is perfectly safe. The Sensor Pad and cord is entirely passive. It carries no electrical current and does not radiate any form of energy. The Nursery Unit emits only very low power radio signals through its antenna. These signals do not present a safety hazard.
- Your monitor can be used in any crib or sleeping area (that has no motion or vibration) designed to safely hold a baby.
- To monitor **twins**, you need one AC201 Movement & Sound Monitor and one AC100 Movement Monitor. The sound monitor of the AC201 is sufficient to monitor both babies in the same room. Having two AC201 Movement & Sound Monitors in one room will cause interference. Do not use the Angelcare® to monitor movement of two babies in one crib or bassinet.

Warnings

- This product is **NOT** a medical device nor a device to prevent cases of Sudden Infant Death Syndrome (SIDS).
- This product is **NOT** a substitute for direct supervision of your baby. Check your baby's activities at regular intervals. Monitoring of premature babies, or those considered to be at risk, should be carried out only under the supervision of a doctor or health professional.
- The Sensor Pad may detect movement from many sources from both inside and outside your baby's room, for example: fans, washing machines, loud music, etc. **Ensure that all vibration sources are eliminated before using the Angelcare® Monitor. Avoid contact with crib when monitor is in use. Refer to the Movement Sensitivity Adjustment and Troubleshooting sections.**
- **ALWAYS** place both units on flat surface in an upright position, away from baby's reach.
- **DO NOT** touch the end of the adapter cord when plugged into an electrical outlet.
- The Angelcare® Monitor uses public airwaves to transmit signals. The Parents Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are in the OFF position when not in use.
- This product should be serviced by qualified personnel. There are no user serviceable parts inside monitor.

Instructions

RETAIN INSTRUCTIONS: Keep user's manual for future reference.

WATER, MOISTURE AND HEAT DAMAGE: Keep all components away from water (sink, bath tub, swimming pool) and places that emit heat (stove, radiator).

VENTILATION: **ALWAYS** place both units on flat surface in an upright position so that air can flow freely around them.

AC POWER SOURCE: Use **ONLY** the AC adapters supplied with the Angelcare® Monitor.

CORD PROTECTION: Avoid damage to the cord of the Sensor Pad and adapters. Route cords so that they are not likely to be walked on or pinched by items placed upon or against them.

CLEANING: Disconnect all units before cleaning. **DO NOT** immerse in water. Use a dry cloth to keep monitor free from dust. **DO NOT** spray with cleaning products or solvents. Wipe Sensor Pad with a very slightly damp cloth with an antiseptic or mild detergent.

OBJECT AND LIQUID ENTRY: Care should be taken so that objects do not fall onto units and that liquids are not spilled into either unit or Sensor Pad through their openings.

NON-USE PERIODS: When not using monitor for long periods, remove the batteries from both units and disconnect the AC adapters from electrical outlet.

Testing Your Angelcare® Monitor

Note: Test your Angelcare® Monitor **BEFORE** first-time use and then on a **periodic** basis.

The Angelcare® Monitor can be used outside of the crib, in a different sleeping area. Test monitor when moving Sensor Pad to a new location to ensure sensitivity is adjusted correctly to the new padding or mattress. Consult the Movement Sensitivity Adjustment section.

Step 1. On the Nursery Unit, switch to the "Sound Only" function. Have someone else speak into the Nursery Unit or place a radio near

Nursery Unit. You should hear sound from the Parents Unit. If there is no sound, refer to the Troubleshooting section. **Make sure that both units are on the same channel, either both on A or B. If reception is not clear, switch to other channel.**

Step 2. Test monitor to detect movement. Switch Nursery Unit to the "Sound & Movement" function. The Movement indicator light will flash red, green and then emit a single "Tic" sound. Place your hand on the mattress. The green light will continue to flash with your every movement. Remove hand from mattress. Due to no movement, a "pre-alarm Tic" sound will emit after 15 seconds. 5 seconds later, the alarm will sound and the red light will flash continuously.

NOTE: If alarm does not sound, the Sensor Pad may be detecting a continuous vibration from the floor, a strong draft or the crib is being touched. **Avoid contact with crib when monitor is in use.** Place crib near a solid supporting wall and away from strong air currents. The sensitivity of the Sensor Pad may need to be reduced, consult the Movement Sensitivity Adjustment section.

Step 3. To stop alarm, place your hand lightly on mattress for Sensor Pad to redetect movement. The red Alarm indicator light on the Nursery Unit will continue to flash 1 minute (Figure 3).

Step 4. Verify that the battery mode is functional by operating **both** units without the AC adapters. The green Power light should appear on both units.

Parents Unit: When orange Low Battery indicator light appears on:

Model AC201: Replace batteries promptly.


Model AC201-R: Recharge Parents Unit for 12 hours.

Nursery Unit: When orange Low Battery indicator light appears a rapid "clicking" is heard, replace batteries promptly. If not, the light will eventually turn red and the monitor will stop working.

Operating Tips

Movement Sensitivity Adjustment

The sensitivity adjustment dial has been preset to suit most situations. However in certain circumstances, the sensitivity may not be ideal. **DO NOT** increase sensitivity unless false alarms occurs and you have followed the advice in the Troubleshooting section for false alarms. Make a note of the pre-set position of the dial should you wish to return to it.

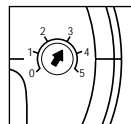The movement indicator light on the Nursery Unit will flash orange when monitor barely detects movement. This could be a result of: baby being in a very deep sleep or baby having moved away from Sensor Pad to edge of crib. If alarm does **NOT** sound, no adjustment of sensitivity is required. If the Movement indicator light flashes orange most of the time with baby directly over the location of the Sensor Pad, a false alarm may occur. If this happens, adjust the sensitivity.

To adjust sensitivity, increase dial to the next setting. Retest monitor.

Channel selection for sound reception

Make sure that both units are on the same channel, either both on A or B. If reception is not clear, switch to other channel. If interference persists, consult the Troubleshooting section.

NOTE: This monitoring system is **NOT** equipped with an out-of-range indicator. No sound will be received if the Parents Unit and Nursery Unit are too far apart.

Yellow Sound Lights

The four Sound Lights allow you to see baby's sounds. The louder the sounds from baby, the more lights appear.

AC power source

Model AC201: The AC adapters should be used whenever possible to extend the battery life. Batteries can be kept in units while using AC power source.

Model AC201-R: Use AC adapter on Nursery Unit whenever possible to extend the battery life. For Parents Unit, use AC adapter **only** to charge NiMH batteries.

Replacing the rechargeable batteries (Model AC201-R only)

When the NiMH batteries in the Parents Unit will not charge or only hold their charge for a short period of time, batteries need to be replaced. **Use only rechargeable batteries in Parents Unit. Do not use Alkaline batteries in Parents Unit.** Plug the Parents Unit into an electrical outlet. **Allow new batteries to charge continuously for 15 hours.**

TROUBLESHOOTING

Problem	Possible Cause	Solution
False alarms.	<ul style="list-style-type: none"> Baby was removed from crib and Nursery Unit was not switched OFF. Sensor Pad cord not connected properly to Nursery Unit. Sensor Pad is not in full contact with crib mattress, or crib mattress is not resting on a completely flat or rigid surface. Baby has moved away from Sensor Pad, to the edge of the crib. 	<ul style="list-style-type: none"> Turn Nursery Unit to the OFF position. Verify the connection between the Sensor Pad and Nursery Unit. Unplug and reconnect the cord. Make sure there is no bedding between mattress and Sensor Pad. Sensor Pad must rest on a completely flat and rigid surface. If your crib has a flexible or uneven base (metal spring base, etc.), place a piece of plywood measuring at least 400 mm X 600 mm X 6 mm (16" X 24" X 1/4") under the sensor pad to create a firm surface. Make sure Sensor Pad is positioned as described in the 5 Easy Steps section. Sensitivity may need to be adjusted, consult the Movement Sensitivity Adjustment section.
Alarm will not sound.	<ul style="list-style-type: none"> Nursery Unit is detecting movement when crib is being touched. Nursery Unit is detecting movement from outside the crib such as motorized devices, strong drafts, etc... Sensitivity is set too high. 	<ul style="list-style-type: none"> Avoid contact with the crib when monitor is ON. Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside crib. You may need to decrease the sensitivity, consult the Movement Sensitivity Adjustment section.
Movement indicator light is flashing orange (monitor is barely detecting movement).	<ul style="list-style-type: none"> Sensor Pad is not in full contact with crib mattress, or crib mattress is not resting on a completely flat or rigid surface. Baby is in a very deep sleep or has moved away from Sensor Pad, to the edge of the crib. 	<ul style="list-style-type: none"> Make sure there is no bedding between mattress and Sensor Pad. Sensor Pad must rest on a completely flat and rigid surface. If your crib has a flexible or uneven base (metal spring base, etc.), place a piece of plywood measuring at least 400 mm X 600 mm X 6 mm (16" X 24" X 1/4") under the sensor pad to create a firm surface. Make sure Sensor Pad is positioned as described in the 5 Easy Steps section. Consult Movement Sensitivity Adjustment section.
No sound is being transmitted.	<ul style="list-style-type: none"> Units are too far apart. Batteries are low or improperly installed. AC adapters are not connected properly or electrical outlet is out of order. One of the units is not ON. Units are set to different channels. 	<ul style="list-style-type: none"> Bring Parents Unit closer to Nursery Unit. Check / replace batteries or recharge Parents Unit (Model AC201-R only). Check connections or change to another electrical outlet. Ensure both units are ON. Make sure both units are on the same channel, either both A or B.
Power ON indicator (green light) does not light up.	<ul style="list-style-type: none"> Batteries are low or improperly installed. AC adapters are not connected properly or electrical outlet is out of order. One of the units is not ON. 	<ul style="list-style-type: none"> Check / replace batteries or recharge Parents Unit (Model AC201-R only). Check connections or change to another electrical outlet. Make sure both units are ON and on the same channel, either both A or B.
Very weak reception.	<ul style="list-style-type: none"> Units are too far apart. One or both units are placed near a large metal mass. One or both units are not in the upright position. Batteries are low. 	<ul style="list-style-type: none"> Bring units closer together. Change the position of one or both units. Place units on flat surface away from baby's reach. Replace the batteries or recharge Parents Unit (Model AC201-R only).
Static, distortion, interference (interference from other monitors, wireless phones, walkie-talkies, etc...).	<ul style="list-style-type: none"> Parents Unit is out of range. Units are set to different channels. Parents Unit located near motorized devices, fluorescent lights, television, etc... 	<ul style="list-style-type: none"> Bring units closer together. Make sure both units are ON and on the same channel, either both on channel A or B. Rotate Parents Unit or move it away from source of interference.
Feedback (loud noises emitted from Parents Unit).	<ul style="list-style-type: none"> Units are too close to each other. 	<ul style="list-style-type: none"> Move units farther apart (at least 3 meters / 10 feet apart). Decrease volume on Parents Unit.

ONE YEAR LIMITED WARRANTY

Angelcare® warrants to the original owner of this product that the Angelcare® Movement & Sound Monitor (Model AC201) and the Angelcare® Rechargeable Movement & Sound Monitor (Model AC201-R) is free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If this monitoring system fails to function properly when used as directed under normal conditions, within the one year period, Angelcare® will either repair or replace the product at our option free of charge, when it is returned complete and in its original packaging, prepaid postage, and insured to Angelcare Monitors, 550 Chemin du golf, suite 202, Nun's Island, Quebec, H3E 1A8.

The product must be accompanied by a proof of purchase, either a bill of sale, receipted invoice, or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a product that has been damaged as a result of improper maintenance, accidents, improper voltage supply or other misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranties are given.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM USE.

IMPORTANT! This Angelcare® Monitor is a personal care product. Please do not return this product to the store.

If you have questions or need help, please call customer service at 1-866-761-0511 before returning this product to the retailer.

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